

# CAWO Plate Cleaner



A dedicated cleaner for CR imaging plates. Protect your investment...



# Solutions

## CAWO Plate Cleaner

An easy-to-use, ethanol based cleaner that ensures longer life and higher performance for CR imaging plates.

### TECHNICAL SPECIFICATIONS

#### Solvent

- Ethanol solution

#### Contains

- 0,5 l

#### Safety

- Highly flammable
- Keep container closed
- Keep away from sources of ignition

Safety datasheets are available on request

### Protect your investment

Dust and dirt build up on CR phosphor plates, leading to image artifacts. To avoid this, plates need cleaning at regular intervals. As CR imaging plates are highly sensitive to water-based cleaning fluids, CAWO has developed an ethanol-based plate cleaner. Using this cleaner, specially designed for CR phosphor plates, will help ensure them the longest service life.


### Simple to use

CAWO Plate Cleaner is simple to use: moisten a piece of gauze with the cleaner, wipe gently across the plate's surface and wait for any remaining solvent to evaporate.

CAWO Plate Cleaner is suitable for use with CR imaging plates only, not with conventional intensifying screens.

### Cleaning frequency


Plates subject to a normal frequency of use should be cleaned at least once every month or whenever there is doubt about particles visible on the X-ray images.

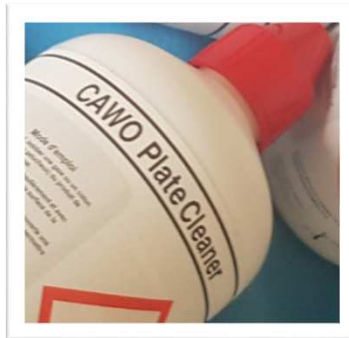

#### Business Contact:

CAWO Solutions  
P.O. Box 11 29  
D - 86521 Schrobenhausen  
Phone: (++49) 8252 9109-21  
Internet: [www.cawo.com](http://www.cawo.com)  
E-mail: [cawo@agfa.com](mailto:cawo@agfa.com)

 **Solutions**

 Agfa NV - Septestraat 27 - 2640 Mortsel - Belgium